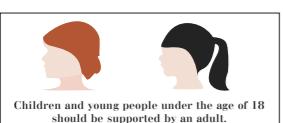
Coronavirus (2019-nCoV)-Antigentest-Instructions for Use

REF

HGCC134S0101 (1T/Kit)
HGCC134S0105 (5T/Kit)
HGCC134S0120 (20T/Kit)
HGCC134S0140 (40T/Kit)

SPECIMEN COLLECTION


COMPONENTS

Instructions for Use

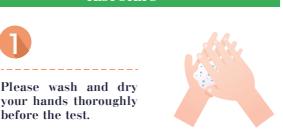
TEST STEPS

Please wash and dry your hands thoroughly before the test.

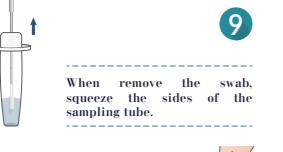Soak the swab in the buffer for at least 15 seconds, stir the swab several times, and squeeze the swab head 3 times.


Repeat sampling with the same swab in the other nostril.

When remove the swab, squeeze the sides of the sampling tube.

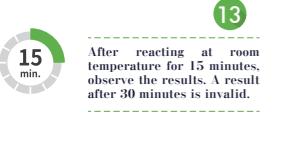Close the sampling tube with the tube lid.

Open the tube lid at the front end of the sampling tube, and drop 4 drops of the treated sample into the sample well (S) of the test cassette.

After reacting at room temperature for 15 minutes, observe the results. A result after 30 minutes is invalid.

Read the Instructions for Use carefully.

Please scan the QR code to watch the operation video.

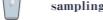
Take the swab out of the package and do not touch the sampling end.

Carefully insert the swab 1.5 cm into the nostril until resistance is felt. Under moderate pressure, swab the surface of the nostril 4 to 6 times for at least 15 seconds.


Repeat sampling with the same swab in the other nostril.


Open the larger end of the sampling tube, and insert the swab after collecting the sample into the sampling tube.

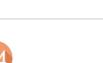
Open the larger end of the sampling tube, and insert the swab after collecting the sample into the sampling tube.

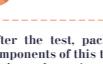Close the sampling tube with the tube lid.

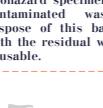Open the foil pouch and place the test cassette on a flat surface.

Wash or disinfect hands again.

INTERPRETATION OF RESULTS

Positive: Two red bands appear in the result window, as shown in Fig. 1: a red or magenta-colored line appears at the C-line and at the T-line, indicating that the antigen test result for the sample is a positive, and there is a suspected infection of COVID-19.

Negative: As shown in Fig. 2, if only a horizontal red line is visible in the control area (C), this can mean that you are negative or that the viral load is too low to be recognized by the test.

My result is negative. What should I do?

You can always test yourself whether you have symptoms or not. Please note that the test result is a snapshot that is valid for this point in time. Tests should therefore be repeated in accordance with the requirements of the responsible authorities.

What do I have to pay attention to in order to get the most exact test result possible?

Always follow the instructions for use exactly. Perform the test immediately after collecting the sample. Put the drops from the sample tube only into the designated well of the test cassette. Dispense four drops from the sample tube. Too many or too few drops can lead to an incorrect or invalid test result.

My result is negative. What should I do?

You can always test yourself whether you have symptoms or not. Please note that the test result is a snapshot that is valid for this point in time. Tests should therefore be repeated in accordance with the requirements of the responsible authorities.

How does the test work?

The N-protein of the SARS-CoV-2 virus reacts with the coating of the test line and leads to a color change, i.e. a red line appears. If the sample does not contain any virus proteins or antigens, no red test line (T) appears.

Can this test cassette be reused or used by more than one person?

This test cassette is for single use and cannot be reused or used by more than one person.

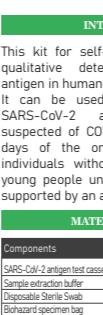
Where do I dispose of the product?

The test kit can be disposed of with normal household waste.

Invalid: As shown in Fig. 3, no line appears at position C (control area) in the result window, indicating that the test is invalid. This can be caused by a possibly incorrect test operation.


Please repeat the test with a new test kit according to the instructions for use.

After reacting at room temperature for 15 minutes, observe the results. A result after 30 minutes is invalid.

INTENDED USE

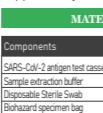This kit for self-testing is used for in vitro qualitative determination of SARS-CoV-2 antigen in human anterior nasal swab samples. It can be used for rapid investigation of SARS-CoV-2 antigen from individuals suspected of COVID-19 within the first seven days of the onset of symptoms and the individuals without symptoms. Children and young people under the age of 18 should be supported by an adult.

MATERIALS PROVIDED

Components	1T/Kit	5T/Kit	20T/Kit	40T/Kit
SARS-CoV-2 antigen test cassette	1 test	5 tests	20 tests	40 tests
Sample extraction buffer	1pc	5pcs	20pcs	40pcs
Disposable Sterile Swab	1 test	5 tests	20 tests	40 tests
Biohazard specimen bag	1pc	5pcs	20pcs	40pcs
Instructions for use	1pc	1pc	1pc	1pc

LIMITATIONS

1. The kit is a qualitative test that cannot quantify the concentration of SARS-CoV-2 antigen.
2. The test result of this kit is not the only confirmation indicator for clinical indications. If the test result does not match clinical evidence, it is recommended to perform additional tests to verify the result.
3. Children and young people under the age of 18 should be supported by an adult.
4. This test cannot be used for non-human samples.

PERFORMANCE

1.Exogenous/Endogenous Interference Substances studies:
There was no interference for potential interfering substances listed below.

(1)Exogenous factor

No.	Exogenous factor	Interfering substances	Test conc.
1	Neaspray or drops	Phenylamine 2 ⁻ 10 TD50/mL	1 test
2	Spray nasal spray 10% (v/v)	Dexamethasone 2 ⁻ 10 TD50/mL	5 tests
3	Nasal corticosteroids	Rhinocort 0.05g/mL	20 tests
4	Nasal corticosteroids	Tramadol acetone 0.05g/mL	40 tests
5	Nasal corticosteroids	Mometasone 0.05g/mL	1 test

(2)Endogenous factor

No.	Exogenous factor	Interfering substances	Test conc.
1	Autimmune disease	Human anti-mouse antibody, HAMA 800 ng/mL	1 test
2	Serum protein	Whole blood anticoagulated (human), EDTA 10% (w/v)	5 tests

CROSS-Reactivity & Microbial interference:

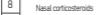
There was no cross-reaction and interference with the potential cross-reacting microorganisms listed below.

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